


HALF SHORT DRESSING INSTRUCTIONS

- DIRECTIONS**
- FOR LEFT-SIDED PROCEDURES, TURN GARMENT INSIDE-OUT
 - LOGO ALWAYS FACES THE FRONT AND ARROW POINTS TO PROCEDURAL LEG

- 1** Garment is packaged for right-sided procedures. For left-sided procedures, turn garment inside-out. Do not release the waistband to turn inside-out or put-on.
- 2** While seated, step into the garment with the outside logo facing the front and the arrow pointing to the procedural leg.
- 3** Pull up garment to waist-level. To adjust, push or pull front panel. Ensure waistband is flat and not twisted.
- 4** If necessary, to tighten or loosen waistband, release side and reattach to fit.

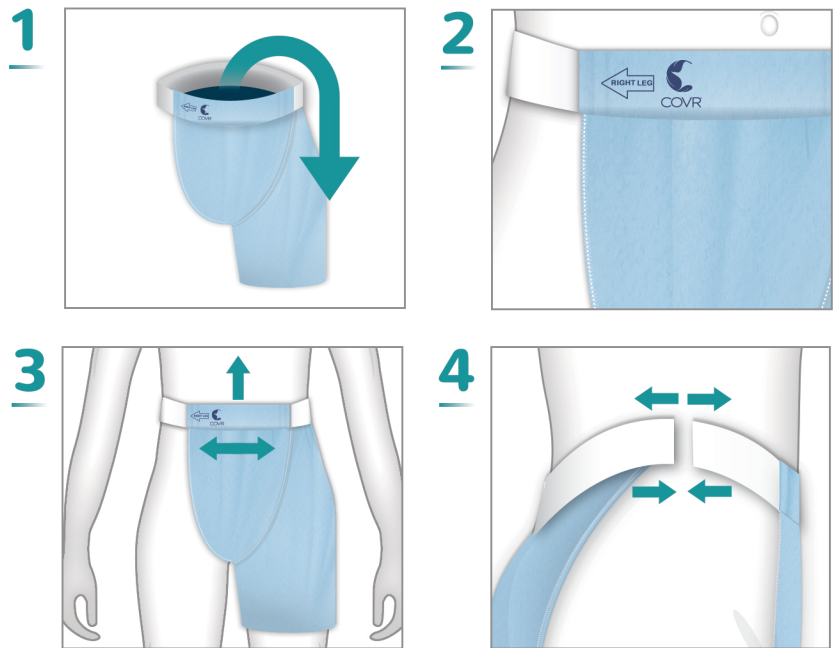

GARMENT VIEW

